

### Step 01

- Ensure the patients hand is clean and warm.
- Hold the finger at a downward angle.
- Only use the lancet provided in the test kit.
- Only use the pipette supplied with the test kit.
- Offer the pipette to the sample – **DO NOT** squeeze the bulb, the pipette will fill via capillary action.
- Ensure the sample reaches the black fill line.
- Ensure there are no bubbles in the sample.
- The finger may be massaged to secure the required volume.



### Step 02

Deliver the whole sample to the cassette sample well by pressing firmly on the pipette bulb.



### Step 03

Add 5-6 drops of diluent, as supplied, to the test well. Wait for 15 minutes and then read the result. Do not read before or after the 15 minute period.



### Step 04

Reading the result



Cassette showing positive result after 15 minutes.



Cassette showing negative result.



Cassettes showing failed control line. Result therefore inconclusive. Repeat the test with a fresh cassette and sample.